

TRC-NW8 series

Non-Mydriatic Retinal Camera



Imaging System


IMAGEnet R4

IMAGEnet R4's cutting edge and comprehensive patient database management system features efficient data viewing, storage, analysis and flexible networking solutions. With its unique utilities, such as Auto Mosaic, stereo functionality, patient consultation, IMAGEnet R4 software streamlines exam analysis and enhances communication with patient.




EZ Capture

Simple & easy software EZ Capture allows you to transfer and save retinal images with intuitive handling. Timer, R/L eye detection, angle and patient data will also be transferred with images.

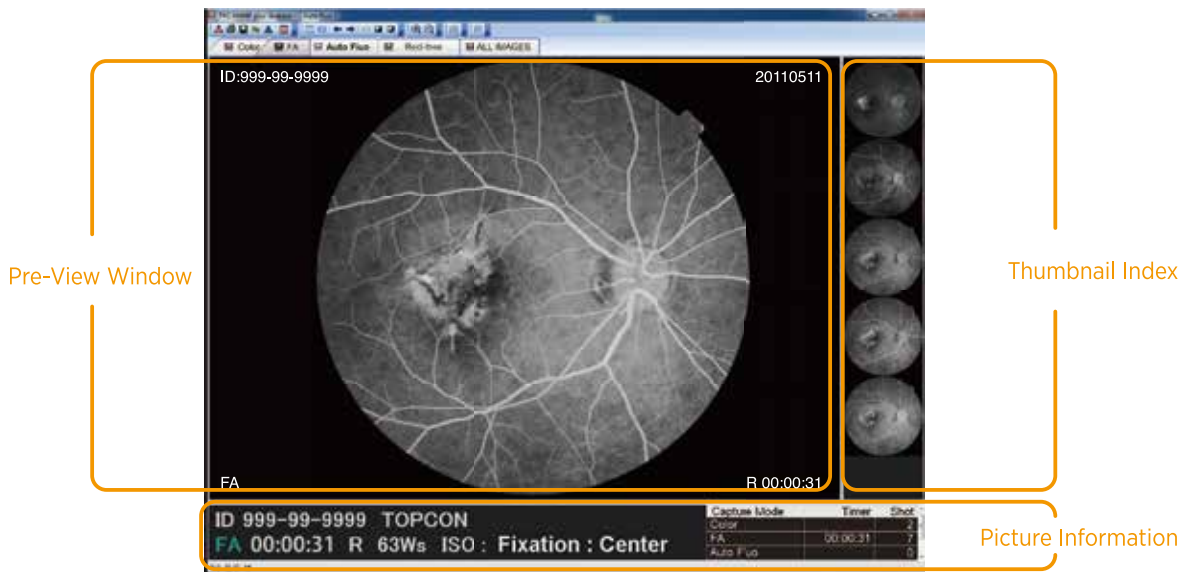

Auto Mosaic Optional Software

A panoramic view of the retina is seamlessly stitched together using Auto Mosaic software. Using a special algorithm to detect vessel edges, fields taken at random are automatically arranged in their correct positions without user intervention.

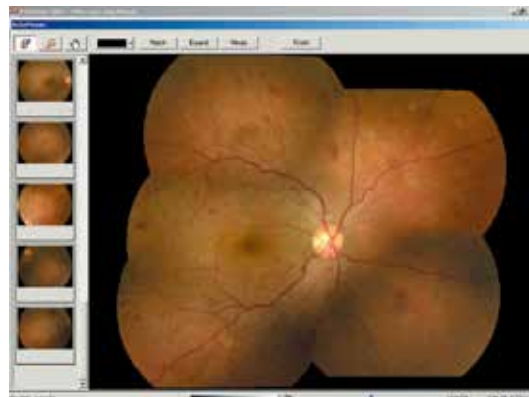

High Quality Image

Quality of Image is Crucial as Well as Durability




With these important requirements Topcon did not hesitate in deciding on which manufacturer it would use for its digital camera supplier. Nikon probably has the largest reputation when it comes to digital SLR cameras and is used by most of the worlds press for very good reason. Therefore you have no need to worry that your new TRC-NW8 series will ever be compromised by the digital SLR in terms of quality of images or durability.



Healthy Eye



Diabetic Retinopathy

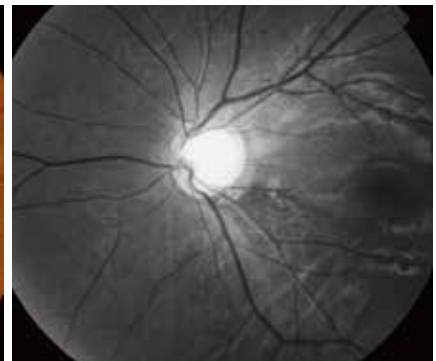


Glaucoma

Stereo Photography



Today there is an increasing need for viewing stereo images and with the TRC-NW8 series camera this has been made very simple with the inclusion of a stereo switch. Once pressed the camera controls fixation to allow for two consecutive images to be captured with just the right amount of shift. These stereo pairs can then be viewed easily on Topcon's IMAGEnet™ software and allow for easier diagnosis.

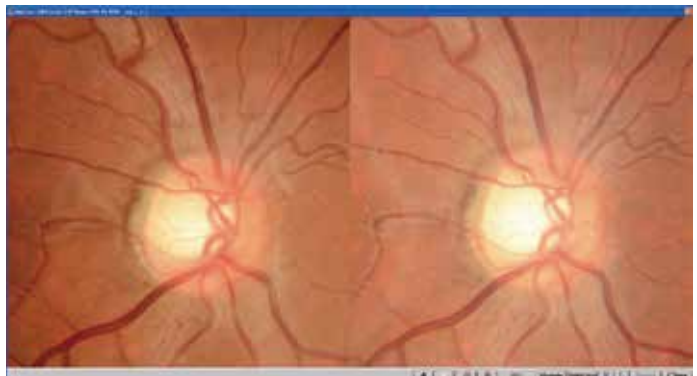


True Optical Red-free Photography



Topcon has developed an innovative feature on TRC-NW8/TRC-NW8F: True optical Red-free capability with an actual Red-free filter. This feature allows for high resolution Red-free images acquisition.

*TRC-NW8F plus :Digital Red-free



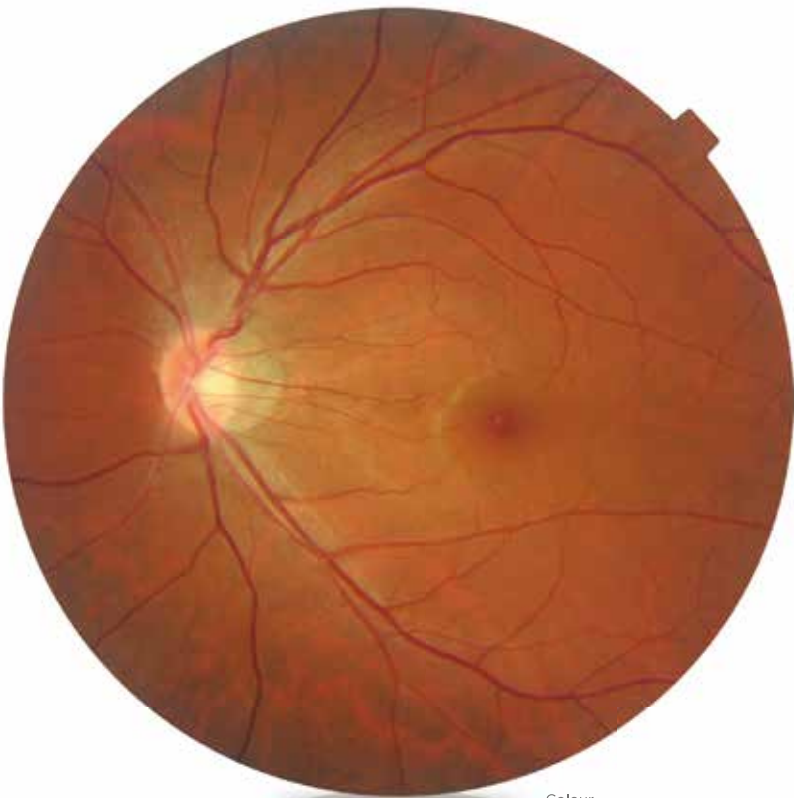

Colour




Red-free

New Combo Retinal Camera TRC-NW8 series




Features

- » Multi Functional Retinal Camera
- » Lower Flash Intensity
- » Easy to Use
- » Continuous Shooting Function
- » Auto Focus & Auto Shoot
- » Auto Small Pupil Detection
- » Compact Design

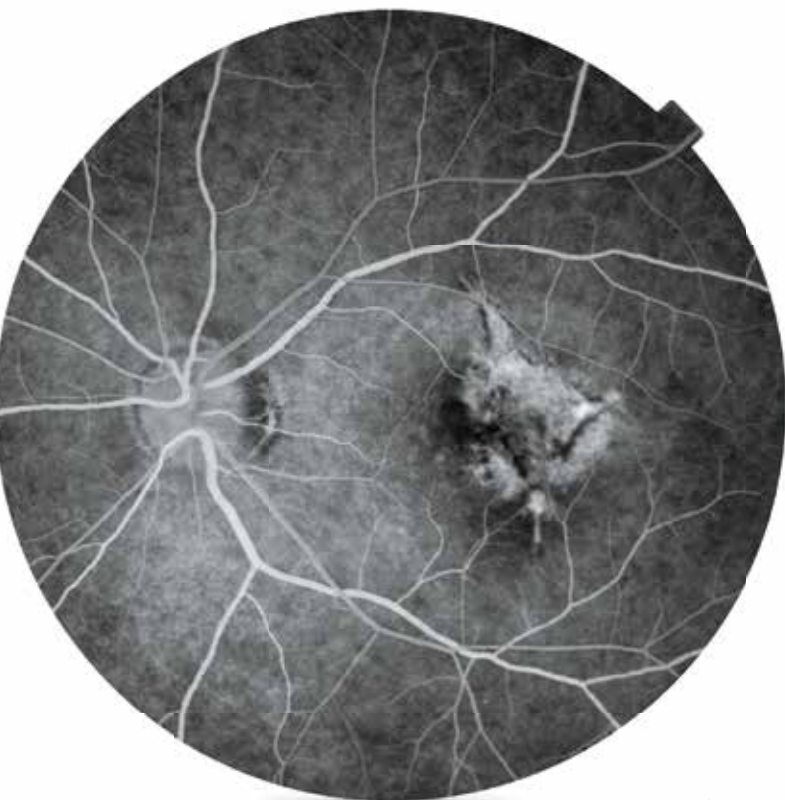

Fluorescence Angiography

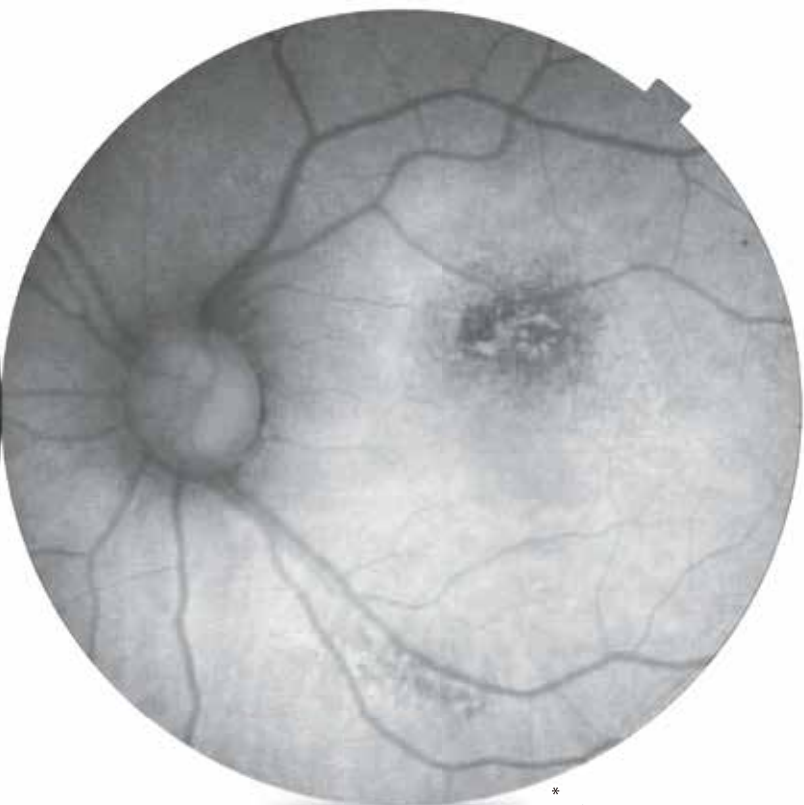

* Fundus AutoFluorescence

Impressive Line-up of Retinal Imaging for All Vision Care Professionals

With TRC-NW8 series, it is possible to capture not only Non Mydriatic Colour / Red-free / Fluorescein Angiography, but also Fundus AutoFluorescein to answer all your many needs.



TRC-NW8 allows easy operation by Auto Shoot / Auto Focus function.



TRC-NW8F allows patient-friendly Fluorescence Angiography photography by lower flash intensity.



TRC-NW8F plus allows to capture multi photography such as Colour, Red-free,** Fluorescence Angiography, and AutoFluorescence photography.

Easy to Use



TRC-NW8 Series
Non-Mydriatic Retinal Camera

Auto Focus & Auto Shoot TRC-NW8 TRC-NW8F TRC-NW8F plus

TRC-NW8 series is Topcon's multi-functional retinal camera which is focused on "simple operation". With our wealth of experience in the production of retinal cameras, we have been able to bring all of the benefits of our non mydriatic camera to this brand new multi functional camera. The ability of Auto Focus and Auto Shoot makes the operation of this camera extremely simple and user friendly. Still allowing manual override when required.

Auto Focus
Split lines are automatically matched.

Auto Shoot
Once alignment dots are made in () area manually, it automatically shoots an image.

※ Exclusively for colour, Red-free and FAF photography


Auto Small Pupil Detection

TRC-NW8

TRC-NW8F

TRC-NW8F plus


Small pupils are a common phenomenon, and when mydriatic drops can not be used, the “Small Pupil” (min. ϕ 3.3mm) option within the TRC-NW8 series, allows you to be able to get a shot.

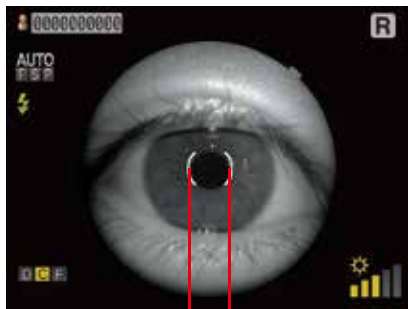

ϕ 3.3mm



Auto Small Pupil Detection Mode:
Angle of coverage 45°

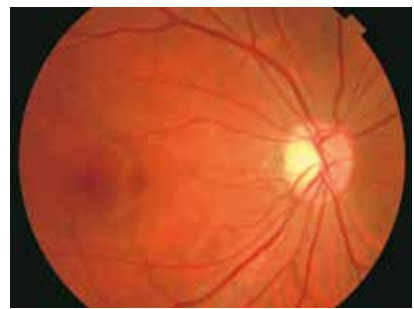

Auto Small Pupil Detection Mode:
Angle of coverage 30°(Digital Zoom)

Low Flash Intensity

TRC-NW8

TRC-NW8F

TRC-NW8F plus

Topcon has always endeavoured to keep flash levels as low as possible while at the same time ensuring that good exposed images are achieved. For FA photography, it can be performed by half the flash intensity compared to our conventional models.



One-touch Operation

Red-free photography is easily switchable only the filter lever, and FA photography is also easily switchable only by pressing the mode switch. In case of TRC-NW8F plus model, simply pulling out the filter lever activates FAF photography mode.



TRC-NW8

TRC-NW8F

※ Optical Red-free



» Red-free Image



TRC-NW8F

TRC-NW8F plus



» Fluorescence
Angiography Image



TRC-NW8F plus



» Fundus
AutoFluorescence



TRC-NW8F plus


※ Digital Red-free

Fluorescence Angiography Photography

TRC-NW8F

TRC-NW8F plus

Fluorescein angiography with TRC-NW8F/TRC-NW8F plus allows for much easier photographic sessions. With the lower flash levels used, patients will find the experience much more comfortable and as a consequence the operators are able to concentrate on getting the images they need.

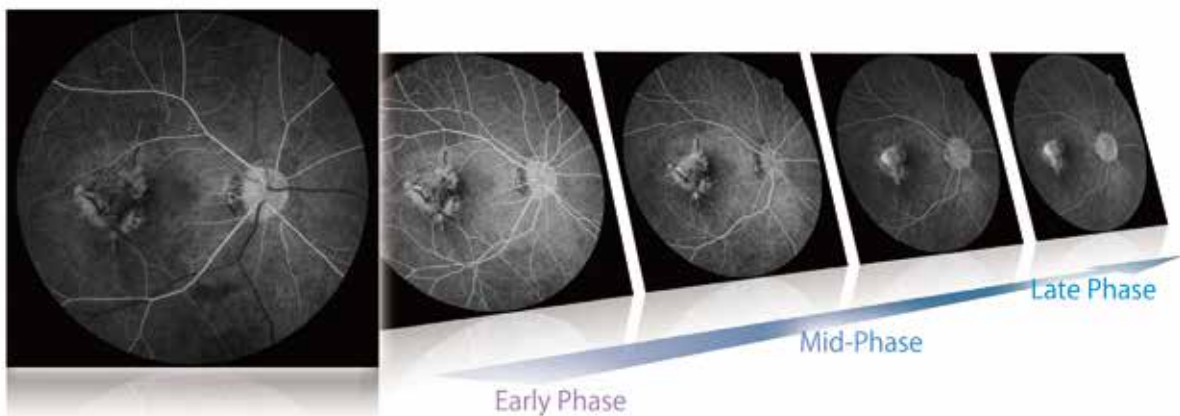

Always High Contrast Images Available

TRC-NW8F

TRC-NW8F plus

Using the TRC-NW8F/TRC-NW8F plus of cameras always provides the operator with the option to use high contrast or a sharpening tool in order to aid diagnosis.

*Incorporated into Topcon optional EZ Capture / IMAGEnet™ Software



Fundus AutoFluorescence Photography

TRC-NW8F plus

Use of the TRC-NW8F plus allows the operator to capture the AutoFluorescence of Lipofuscin and to show the sub macula area clearly using the in-built FAF filters. These filters have been specifically designed to avoid absorption of xanthophy. It is very useful to observe the change of chorioretinal atrophy in age-related macular degeneration.

* Spaide FAF filter is produced under the supervision and authorization of Dr. Richard F. Spaide, Vitreous, Retina, Macula Consultants of New York, NY, USA.



Specifications

Angle of coverage	45°
Working distance	40.7mm
Pupil diameter for photography	φ 4.0mm or more φ 3.3mm or more when the small pupil diaphragm is used
Type of photography	Colour photography, Red-free photography [*] , FA photography ^{**} and FAF photography ^{***}
Patient dioptrre correction range	Without correction lens: -13D to +12D (where split lines are used) With minus correction lens: -33D to -12D With plus correction lens: +9D to +40D
Auxiliary function for photography	Auto focus function (Used only in the split line working range. This can be turned ON / OFF)
Fixation Target	Internal / External fixation target can be selected Internal fixation target Centre / Periphery Right / Left eye automatic detection Optional position presetting function
Base movement	Back & forth:46mm, Right & left:100mm, Up & down:30mm
Chinrest movement	67mm
Power source	Frequency : 50 / 60Hz Voltage : AC100-240V
Weight	23.5kg (TRC-NW8/TRC-NW8F), 23.8kg (TRC-NW8F plus)
Dimensions	274(W) X 508(D) X 536-566(H)mm
Power consumptions	400VA(Maximum), 100VA(Normal)

* TRC-NW8F plus model: Digital Red-free

**Only for TRC-NW8F and TRC-NW8F plus models

*** Only for TRC-NW8F plus model

Optional Accessory



External Fixation Target

IMPORTANT In order to obtain the best results with this instrument, please be sure to review all user instructions prior to operation. Not all products, services, or offers are available in all markets. Contact your local distributor for country-specific information and availability.

TOPCON INSTRUMENTS (MALAYSIA) SDN. BHD.

(Regional Office for Topcon Healthcare Southeast Asia)

Unit 2, 4, Jalan Pensyarah U1/28, Hicom-glenmarie Industrial Park,
40150 Shah Alam, Selangor, MALAYSIA
Phone: +603-766 16260 Fax: +603-766 16261
Email : mys_tim_marketing_sm@topcon.com
www.topconhealthcare.my

TOPCON SINGAPORE MEDICAL PTE. LTD.

100G Pasir Panjang Road, #02-18, Interlocal Centre,
SINGAPORE 118523
Phone: +65-68720606 Fax: +65-67736150
E-mail: med.sales.sg@topcon.com
www.topconhealthcare.sg

TOPCON INSTRUMENTS (THAILAND) CO., LTD.

77/162 Sinnsathorn Tower, 37th Floor, Krungthonburi Rd.,
Klongtongnai, Klongsarn, Bangkok 10600, THAILAND
Phone : +66-02-440-1152 Fax : +66-02-440-1158
Email : tha_medical@topcon.com
www.eyecare.topcon.co.th

MEHRA EYETECH PRIVATE LIMITED

801 B Wing, Lotus Corporate Park, Graham Firth Steel Compound
Goregaon (East) Mumbai 400063 Maharashtra, INDIA
Phone: +91-22-61285455
E-mail: sales@mehraeyetech.in
www.topconhealthcare.in

TOPCON (BEIJING) MEDICAL TECHNOLOGY CO., LTD.

Room 2808, Tower C, JinChangAn Building, No.82, Middle Section of
East 4th Ring Road, Chaoyang District, Beijing 100124, P.R. CHINA
Phone: +86-10-8794-5176
E-mail: cn_marketing@topcon.com
www.topcon-china.net/

TOPCON KOREA MEDICAL CO., LTD.

2F YK Building, 205, Dogok-ro, Gangnam-gu,
Seoul, Republic of Korea
Phone : +82-2-6959-7947
E-mail: tkm@topcon.com
www.topconhealthcare.kr/



TOPCON CORPORATION

75-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8580, JAPAN.
Phone: +81-(0)3-3558-2522/2502
Fax: +81-(0)3-3965-6898
www.topconhealthcare.jp

