
PC-50SB

Pixel Chart

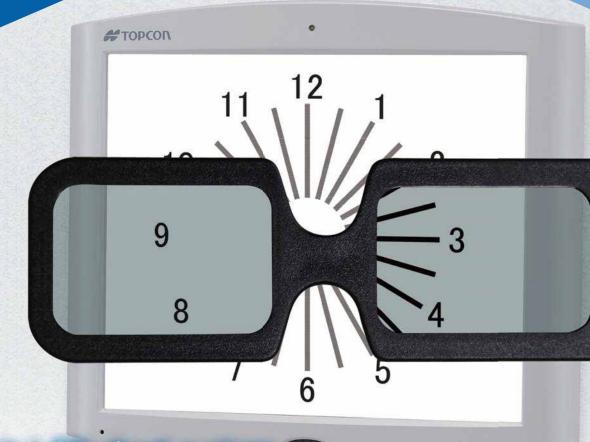

Innovation in LCD chart system by Circular Polarization



Pixel Chart PC-50SB

New LCD chart monitor with innovative "CIRCULAR POLARIZATION" method



Features

- >> Circular polarization
- >> Open binocular vision test
- 3D contents
- >> variety of chart
- >> variety of contents
- » maintenance free

Open binocular vision test with Circular Polarization method

What is "Circular Polarization"?

Circular polarization is latest visual method to realize 3D TV or 3D movies. By wearing the special circular polarization glasses, the polarization will not be effected even if the face is inclined. Therefore binocular balance test with polarization chart can be done without customer's making effort to keep their face straight.





(looking with face inclining)

What is "open" binocular vision test?

When an eye is tested, you do not have to use occluder or fogging on the other eye, thanks to circular polarization filters. The customer sees the chart with one eye and it disappeared with the other eye. Open binocular refraction will reduce the unusual accommodation.

Conventional refraction

With one eye occluded, the other open eye is examined.




9 8 7 8 5

Vision in mind

New! open binocular refraction

Polarization disappears the chart to one eye, and the other eye is examined.

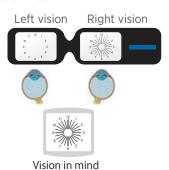

Interactive Application of Variety of Contents

Expanded Application of KB-50S for Your Daily Scene

Needless to say that the KB-50S functions as a remote controller for chart switching for visual acuity tresting it incorpotates a new function for near view simulation.

>> Daily Scene

Various contents of daily scene are ready for near view with KB-50S and far view with PC-50S. This combination is quite useful for checking the final prescription.






3D Contents

In light of the ever increasing use of 3D technology in today's world, we have incorporated high-tech 3D content into the new PC-50SB. 3D movies and images can be projected on the PC-50SB, which allows you to check vision in 3D and keep up with the latest technology.



Active Chart

TOPCON's unique "Active Charts" helps testing correct visual funvtion in practical condition.

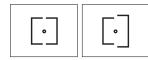

Coincidence Test (Aniseikonia Test)



Phoria Test



Phoria Test






Astigmatism Test

Contrast Test

It helps to examine the contrast sensitivity with piual and singular frequency chart.

Change of contrast Change of contrast Change of contrast Single ratio character contrast Change of contrast Plural ratio character contrast

Black-and-white inversion

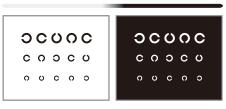


Specifications

Test distance	Standard 5m
Test Chart	» Optometry
	Landolt ring, numerals, alphabets,
	tumbling E, pictures, screening
	» Visual function
	Red / green test, astigmatism, heterophoria,
	stereoscopic vision, coincidence (aniseikonia),
	binocular vision.
Dimensions	460mm(W) x 65mm(D) x 392mm(H)
Power supply	AC100-120V / 220-240V
	50-60Hz
Weight	7.5kg

System Chart Compu Vision . 00 . Power Supply Unit w/External PC Power Supply Unit w/Built in PC 1 Dial Controller PC Mouse & Monitor Mouse & Monitor **KB-50S** Wave-Front -Computerized Lensmeter 1 Pixel Chart PC-50SB EZ Meter Auto Chart Mirror Chart Projector Auto Refractometer Auto Kerato-Refractometer

Chart Selection

A, B, F, H1 types of standard chart set are included.

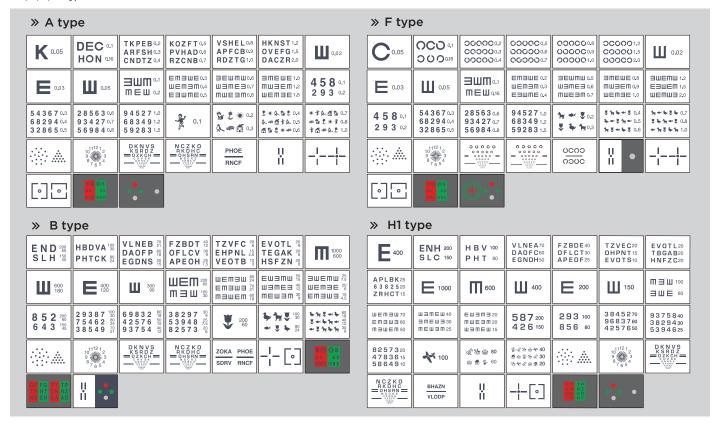

IMPORTANT In order to obtain the best results with this instrument, please be sure to review all user instructions prior to operation

TOPCON INSTRUMENTS (MALAYSIA) SDN. BHD.

(Regional Office for Topcon Healthcare Southeast Asia)
Unit 2, 4, Jalan Pensyarah U1/28, Hicom-glenmarie Industrial Park
40150 Shah Alam, Selangor, MALAYSIA
Phone: +603-766 16260 Fax: +603-766 16261
Email: mys_tim_marketing_sm@topcon.com
www.topconhealthcare.my

TOPCON SINGAPORE MEDICAL PTE. LTD.

100G Pasir Panjang Road, #02-18, Interlocal Centre, SINGAPORE 118523 Phone: +65-68720606 Fax:+65-67736150 E-mail: med.sales.sg@topcon.com www.topconhealthcare.sg

TOPCON INSTRUMENTS (THAILAND) CO., LTD.

77/162 Sinnsathorn Tower, 37th Floor, Krungthonburi Klongtonsai, Klongsarn, Bangkol 10600, THAILAND Phone: +66-02-440-1152 Fax: +66-02-440-1158 Email: tha_medical@topcon.com www.eyecare.topcon.co.th

MEHRA EYETECH PRIVATE LIMITED

REHKA ETETECH PRIVATE LIMITED

801 B Wing, Lotus Corporate Park, Graham Firth Steel Compound
Goregaon (East) Mumbai 400063 Maharashtra, INDIA
Phone: +91-22-61285455
E-mail: sales@mehraeyetech.in
www.topconhealthcare.in

TOPCON (BEIJING) MEDICAL TECHNOLOGY CO., LTD.

ROM 2808, Tower C, InChangAn Building, No.82, Middle Section Rom 2808, Tower C, InChangAn Building, No.82, Middle Section East 4th Ring Road, Chaoyang District, Beijing 100124, P.R. CHINA Phone: +86-10-8794-5176 E-mail: cn_marketing@topcon.com www.topcon-china.net/

TOPCON KOREA MEDICAL CO., LTD.

2F YK Building, 205, Dogol Seoul, Republic of Korea Phone:+82-2-6959-7947 E-mail:tkm@topcon.com www.topconhealthcare.kr/





TOPCON CORPORATION

75-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8580, JAPAN. Phone: +81-(0)3-3558-2522/2502 Fax: +81-(0)3-3965-6898 www.topconhealthcare.jp

